STORZ

KARL STORZ—ENDOSKOPE

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1 Important Information

It is recommended to check the suitability of the product for the intended procedure prior to use.

Thank you for your expression of confidence in the KARL STORZ brand name. Like all of our other products, this product is the result of years of experience and great care in manufacture. You and your organization have decided in favor of a modern, high-quality item of equipment from KARL STORZ.

This instruction manual is intended to serve as an aid in the proper handling and cleaning of the C-MAC® Monitor 8403 ZX. All required details and all actions on your part are clearly presented and explained. We ask you to read this manual carefully and keep available for ready reference.

 **NOTE:** All users must have been instructed in the operation and use of the C-MAC® Monitor 8403 ZX, otherwise this monitor may not be used.

 **NOTE:** The C-MAC® Monitor 8403 ZX must be used in accordance with the airway algorithm in the respective country of application or the official specifications for airway management.

 **WARNING:** KARL STORZ instruments and equipment are not sterile when delivered, and must therefore be cleaned and disinfected prior to initial use and prior to every subsequent use.

KARL STORZ is constantly working on the further development of all products. Please appreciate that changes to the scope of supply in form, equipment and technology are possible for this reason. Therefore, no claims may be derived from the information, figures, and descriptions in this manual.

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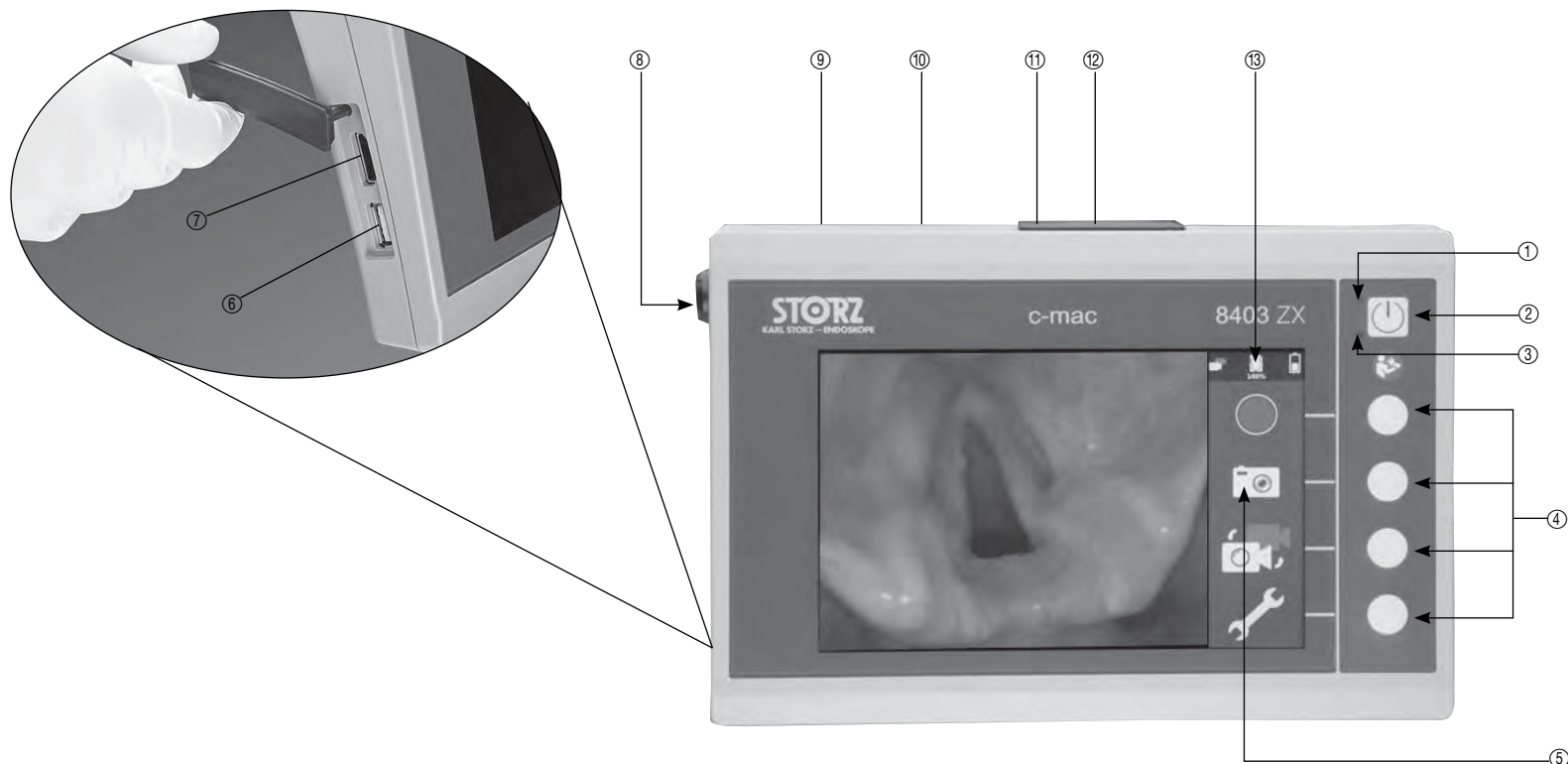
- ① Pilot lamp (green)
- ② On/Off switch
- ③ Pilot lamp (orange)
- ④ Control keys
- ⑤ Function symbols
- ⑥ USB port
- ⑦ HDMI port
- ⑧ Connection for CMOS videoscope 2
- ⑨ Connection for CMOS videoscope 1
- ⑩ Mains connection
- ⑪ Reset button (see page 29)
- ⑫ Holder for SD memory card
- ⑬ Status bar

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Controls, connectors and their uses

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External system components

Charger for C-MAC®

CAUTION: Damage to C-MAC® Monitor. Only the charger ET27-30-0004943 may be used. The use of other chargers can result in malfunctions or damage to the monitor.

Using the medically approved charger (ET27-30-0004943), the Li-Ion battery of the monitor can be recharged within a very short time.

Moreover, using the charger, the monitor can also be operated when the Li-Ion battery is discharged.

The charger has a green light indicator (14), which indicates the presence of the correct output voltage.

Country-specific configuration

The charger has exchangeable primary adaptors with which it can be adapted to suit the respective power sockets.

- a** Primary adaptor for EUROPE: ET27-30-0004 516
- b** Primary adaptor for UK: ET27-30-0004 517
- c** Primary adaptor for US/Japan: ET27-30-0004 518
- d** Primary adaptor for Australia: ET27-30-0004 519
- e** Primary adaptor USA ET27-30-0004 520
- f** Adaptor cable ET27-30-0004 370

Press the button on the power supply unit to change the adaptor (fig. 1). This releases the adaptor which can then be lifted out.

When inserting a new adaptor, this must be placed in the guide (fig. 2) and pushed toward the cable until there is an audible click (fig. 2, see arrow).

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Control of pollution caused by electronic information products (China RoHS)

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Important: Accompanying documents must be observed

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Do not use the product if the packaging has been damaged

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Read accompanying documents before use

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WEEE symbol: Labeling of electrical devices

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Labeling of batteries/accumulators

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UL approval mark, valid in the USA and Canada

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Degree of protection in accordance with International Protection Codes: Protection against contact with tools or similar and small foreign bodies, protection against vertically dripping water

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
Rated ambient temperature/transducer temperature class

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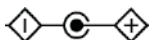1. Important information	II
Pictures of the instruments	III
External system components	IV
Symbol description	V
2. Safety instructions	2
2.1 Intended use	2
2.2 User qualification	2
2.3 Compatibility	3
2.4 Symbols and hazard information	4
3. Commissioning	8
3.1 Commissioning of the monitor with a CMOS videoscope	8
3.1.1 Primary camera	8
3.1.2 Secondary camera	8
3.2 Power supply unit	9
3.3 Use of the HDMI output	9
3.4. Insertion of an SD memory card	10
3.5. Use of the USB port	10
3.6. Switching on the monitor	11
3.7 Monitor status bar	11
3.8 Monitor function keys	14
3.9 Switching off the monitor	27
3.10 Energy management	27
3.11 Fixing the monitor to a stand	29
3.12 Performing a reset	29
3.13 C-MAC® bag 8403 YD	30
4. Cleaning and disinfection	31
5. Maintenance	32
5.1 Maintenance and safety check	32
5.1.1 Maintenance	32
5.1.2 Safety check/repeat inspection according to IEC 62353	32
5.2 Servicing and repair	34
5.3 Limitation of liability	34
5.4 Warranty	34
5.5 Directive compliance	35
5.6 Standard compliance	35
5.7 Battery	36
5.8 Disposal	36
5.9 Repair program	36
5.10 Important information	36
6. Technical description	37
6.1 Technical data	37
6.2 Troubleshooting	39
7. Appendix	42
Accessories	42
Information on electromagnetic compatibility (EMC)	43

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2 Safety instructions

2.1 Intended use

The C-MAC® Monitor is used together with a CMOS videoscope to play back images during an endoscopic intervention.

Contraindications: No contraindications relating directly to the product are currently known. Use of the monitor 8403 ZX is contraindicated if, in the opinion of a responsible physician, the health of the patient is endangered through its use, e.g., due to the general condition of the patient, or if the intervention as such is contraindicated.

2.2 User qualifications

The C-MAC® Monitor 8403 ZX may only be used by persons with an appropriate medical qualification and who are acquainted with the technique. The information given in these instructions only serves to instruct in the correct handling and cleaning of the C-MAC® Monitor.

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NOTE: The recommended camera default settings can be made via the function keys in the factory settings menu (see page 20).

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2.4 Symbols and hazard information

Please read this manual and follow its instructions carefully. The words Warning, Caution, and Note convey special meanings. Wherever they are used in this manual, the following text should be carefully reviewed to ensure the safe and effective use of this device. To make these words stand out clearly, they are accompanied by a pictogram.

WARNING: A Warning indicates that the personal safety of the patient or personnel may be endangered. Failure to observe a Warning could result in injury to the patient or personnel.

CAUTION: A Caution indicates that particular procedures or precautions must be followed to avoid possible damage to the instrument set.

NOTE: A Note indicates special information regarding operation or clarifies important information.

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WARNING: Combinations of medical devices are only assured to be safe if

- they are identified as such in the respective instruction manuals or
- the intended use and interface specifications of the devices used in combination permit this.

WARNING: Check the C-MAC® Monitor 8403 ZX prior to each use to ensure that it functions correctly and has been properly cleaned and/or disinfected.

WARNING: If damage is evident, the C-MAC® Monitor must no longer be used, or sent for repair if appropriate.

WARNING: Modifications to the device are not permitted.

WARNING: Before each use or after a change of viewing modes/settings, the operator should check to ensure the view observed through the endoscope provides a live image (rather than a stored one) and has the correct image orientation.

WARNING: Interconnections conditions require the applied parts of other ME equipment used within the configuration for endoscopic application to be type BF applied parts or type CF applied parts.

WARNING: When energized endoscopes are used with energized endotherapy devices, patient leakage currents may be additive. This is particularly important if a type CF applied part endoscope is used, in which case a type CF applied part energized endotherapy device should be used in order to minimize total patient leakage current.

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WARNING: Before each use, the compatibility of the endoscopic equipment with any accessories and/or energized endotherapy devices should be checked according to any criteria for safe use defined in the instructions for use.

CAUTION: To avoid damaging the monitor, the SD memory card or connecting cable must not be removed whilst video recording.

CAUTION: To avoid damaging the monitor, the SD memory card may only be replaced when the monitor is switched off.

CAUTION: The C-MAC® Monitor 8403 ZX must not be cleaned in an ultrasound bath.

CAUTION: The C-MAC® Monitor 8403 ZX must not be steam-sterilized.

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CAUTION: Maintenance work on the C-MAC® Monitor may only be performed by persons authorized by KARL STORZ.

CAUTION: The battery may only be replaced by persons authorized by KARL STORZ.

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NOTE: It may be more difficult to read the display in bright light, e.g., in direct sunlight.

NOTE: Additional equipment connected to medical electrical equipment must be able to prove their compliance with the respective IEC or ISO standards (e.g., IEC 60950 for data processing equipment). Furthermore, all configurations shall comply with the standard requirements for medical systems (see IEC 60601-1-1 or Clause 16 of the 3rd Ed. of IEC 60601-1, respectively). Anybody connecting additional equipment to medical electrical equipment is a system configurator and is therefore responsible for the system's compliance with the standard requirements for systems. Please note that local laws take priority over the above-mentioned standard requirements. If in doubt, please consult your local specialist dealer or the technical service department. (Standard / directive references: IEC 60601-1+A1+A2:1995: 6.8.2.c, 19.2.b, 19.2.c, IEC 60601-1:2005: 7.9.2.5, 8.1, 16.2.d, MDD 93/42/EEC: Annex I clause 13.6.c).

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3 Commissioning

3.1 Commissioning of the monitor with a CMOS videoscope

Plug the connecting cable of the CMOS videoscope into the connection socket ⑨ of the monitor.

The connecting cable can be connected or disconnected during operation.

The instruction manual of the respective CMOS videoscope must be observed.

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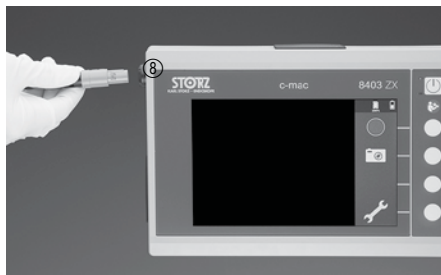
3.1.1 Primary camera

The Monitor 8403 ZX enables two CMOS video units to be connected with Lemo connectors. On the rear of the device, next to the power supply connection, there is a connection socket ⑨ for the primary CMOS video unit.

3.1.2 Secondary camera

The connection ⑧ for the second CMOS video unit is located on the top left-hand side of the housing.

NOTE: All CMOS videoscopes from KARL STORZ can be connected to both connections (see overview table on page 3). Video units from other manufacturers cannot be connected.

NOTE: If two cameras are connected, the user must select in the menu the camera for which the images are to be displayed (see page 16).

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3.2 Power supply unit

Connect the power cord to the monitor's connection socket ⑩ and plug the power supply unit into the mains socket.

The power cord can be fixed to the connection using the union nut.

The battery is charged automatically as soon as the monitor is connected to the power supply. This is displayed by the lighting up of the orange-colored LED ③.

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3.3 Use of the HDMI output

The device features a port for an external monitor. An HDMI cable can be used to send the video signal to an external monitor.

CAUTION: The C-MAC® Monitor must be switched off when an external monitor is connected.

NOTE: The connected monitor must support a resolution of 1024 × 768 or 1280 × 720.

NOTE: To connect the C-MAC® Monitor to a KARL STORZ monitor, the HDMI-DVI adaptor with article number 209190 10 is required.

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3.4 Insertion of an SD memory card

If single images or a video stream are to be saved, then an SD memory card must be inserted into the card holder ⑫.

- ① **NOTE:** To ensure quick documentation, only use SD or SDHC cards of class 4 or higher.
- ① **NOTE:** Before use with the Monitor 8403 ZX, personal SD memory cards must be formatted on the PC with FAT (standard) and may only have one partition.
- ① **NOTE:** The monitor must be switched off when the SD memory card is inserted.

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3.5 Use of the USB port

The USB port serves exclusively to transfer data from the SD memory card to the USB stick. Only USB-compatible storage media may be connected.

- ① **NOTE:** Before use with the Monitor 8403 ZX, USB sticks must be formatted on the PC with FAT (standard) and may only have one partition.
- ① **NOTE:** In the case of large video files or with multiple recordings, a lot of time may be required for the data transfer.
- ① **NOTE:** A USB stick with a large storage capacity can delay the storage of images.
- ① **NOTE:** The video stream can only be viewed on a PC if an MPEG-4 codec is installed on the PC.

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3.6 Switching on the monitor

Switch on the monitor by pressing the power switch ② (the green pilot lamp ① lights up).

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3.7 Monitor status bar

The charge state of the battery, the amount of available storage space of a connected SD memory card or of a connected USB stick as well as, if relevant, the connection of a second camera are all displayed in the monitor status bar ⑬.

The symbols mean:

Connection of a second camera

Displays the currently active camera (only active if two cameras are connected)

Battery charge state

Battery empty

25%: Approx. 30 minutes of operating time

50%: Approx. 1 hour of operating time

75%: Approx. 1.5 hours of operating time

100%: Approx. 2 hours of operating time

Battery is being charged

Replace battery: Maximum battery capacity is less than 50% of the original maximum capacity

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3.7.1 Notifications

The monitor provides additional notifications in certain critical situations.

When a new notification appears, the relevant area in the status bar is highlighted white and a symbol appears in the bottom right-hand corner. The notification can be confirmed with the relevant button. The symbol disappears and the wrench button is released again.

The notification remains visible in the status bar until the status has returned to normal.

Up to 3 notifications can appear simultaneously, these can be confirmed in succession.

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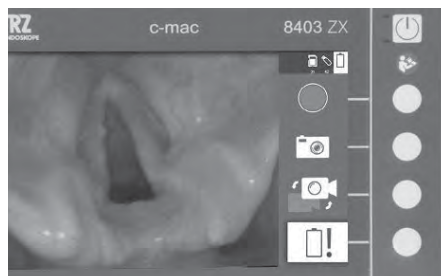
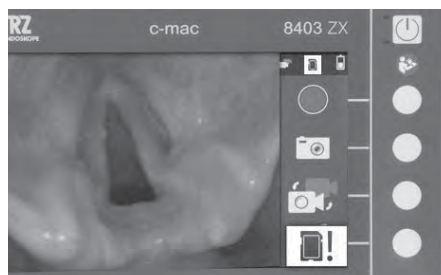
Critical battery charge state

If the battery charge state reaches a critical level, the battery symbol in the status bar is highlighted white. At the same time, in the bottom right-hand corner of the display, an empty battery symbol also appears against a white background together with an exclamation mark.

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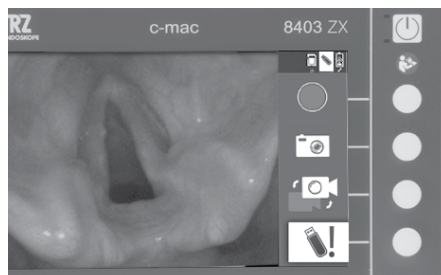
SD memory card full

If the SD memory card inserted in the holder ⑫ is full or write-protected and no more data can be saved, the SD memory card symbol in the status bar is highlighted white. At the same time, in the bottom right-hand corner of the display, a full SD memory card symbol also appears together with an exclamation mark.

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No data can be saved to the USB stick

If the USB stick connected to connection ⑥ does not allow data to be transferred from the SD memory card, the USB stick symbol in the status bar is highlighted white. At the same time, in the bottom right-hand corner of the display, a full USB stick symbol also appears together with an exclamation mark.

Data transfer from the SD memory card to the USB stick is only possible if there is space on the USB stick for the SD memory card data, if the USB stick is not write-protected and if the file system on the USB stick is not damaged.

NOTE: When transferring the data from the SD memory card to a USB stick, the data is copied from the SD memory card and not deleted. The data on the SD memory card must be deleted manually on the monitor (see page 17/18).

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3.8 Monitor function keys

The monitor is equipped with 4 softkeys. These can be used to activate the functions displayed as symbols on the screen.

NOTE: The **video recording** and **image storage** symbols are only visible if an SD memory card is inserted. The **change the camera unit** symbol is only visible if two video units are connected.

NOTE: If the camera has control keys and these keys are activated, certain functions can be performed using these keys.

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3.9 Switching off the monitor

Standby mode: Pressing the power switch ② briefly switches the monitor to standby mode. This allows the monitor to maintain a high state of readiness in the event of brief interruptions. Flashing of the green pilot lamp ① above the power switch indicates that the device is in standby mode.

NOTE: After 20 minutes of keyboard inactivity the monitor switches to standby mode.

After 5 minutes, the system switches itself from standby mode to off mode.

Switching off: Pressing the power switch ② for an extended period (the power switch is held down until the progress bar in the menu bar is complete) switches the monitor off. The green pilot lamp ① goes out.

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3.10 Energy management

The charge state of the battery is displayed on the monitor by the charge indicator in the menu bar.

The symbols mean:

-  Battery empty
-  25%: Approx. 30 minutes of operating time
-  50%: Approx. 1 hour of operating time
-  75%: Approx. 1.5 hours of operating time
-  100%: Approx. 2 hours of operating time
-  Battery is being charged
-  Battery must be replaced (Maximum battery capacity is less than 50% of the original maximum capacity)

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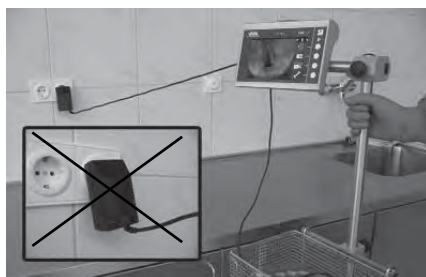
NOTE: The C-MAC® Monitor can be powered using either mains electricity or the integrated battery.

The battery is charged automatically as soon as the monitor is connected to the power supply. This is displayed by the lighting up of the orange-colored LED.

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NOTE: The monitor should be recharged after each use or, at the latest, once the charge state has dropped below 50%.

CAUTION: Damage to the power cord: To remove the power cord from the socket, pull only on the plug housing, never on the cord itself!

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The lithium-ion battery can be used for approx. 2 hours when charged.

The discharged battery is fully charged after 5 hours (when the device is switched off). During the charging time, the orange-colored LED lights up.

If the system is not used for 12 weeks or longer, you should check the charge state using the progress bar described on page 11 and, if necessary, charge.

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3.11 Fixing the monitor to a stand

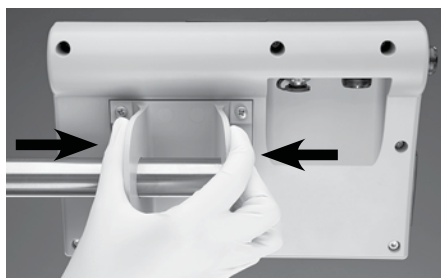
Attach the stand holder to the rear of the monitor using 4 screws.

The VESA 75 holder (8401 YC), which is included in the scope of supply, is suitable for securing devices to tubes with a diameter of 25 mm.

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Press the holder together to allow the stand bar to be pushed through the openings.

Release the holder in the desired position. The selected position is fixed by the holder's basic tension.

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The optionally available clamp 8401 YB (see accessories) can also be secured to the VESA 75 holder and used to attach the monitor to tubes with a diameter of less than 25 mm and/or to rails with a cross section of less than 25 mm.

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3.12 Performing a reset

If a system error occurs (device no longer reacts although connections and power supply are in order), a system reset must be carried out. This means rebooting the system. For this, a ballpoint pen or similar is inserted into the round opening next to the SD memory card holder ⑪ and carefully pushed downwards in order to activate the reset button.

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A reset can alternatively be performed directly by pressing the On/Off button ②. To this end, the On/Off button ② must be held down for at least 10 seconds.

In both cases, the device must then be switched on again using the On/Off button ②.

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3.13 C-MAC® bag 8403 YD

The optionally available protective bag 8403 YD facilitates the mobile use of the C-MAC® Monitor.

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4 Cleaning and disinfection
! **CAUTION:** The manual ‘Cleaning, Disinfection, Care and Sterilization of KARL STORZ Instruments’ must be followed. The cleaning, disinfection and sterilization procedures are explained therein in detail. The list of products approved by KARL STORZ can be found on the Internet at ‘www.karlstorz.com’.

Manual wipe-down disinfection

The design allows only for wipe-down disinfection. The exterior surfaces of the medical device must be wiped clean with a disposable cloth moistened with disinfectant or a ready to use soaked disinfectant cloth. Due to their protein-fixating effect and material incompatibility, alcohol-based agents must not be used. The chemical manufacturer's specifications regarding material compatibility must be observed.

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✓ suitable procedures
---- unsuitable procedures

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5.1 Maintenance and safety check

5.1.1 Maintenance

Preventive maintenance is not essential. Regular maintenance can, however, contribute to identifying potential problems before they become serious, thus enhancing the instrument's reliability and extending its useful service life. Maintenance services can be obtained from your local representative or from the manufacturer.

5.1.2 Safety check/repeat inspection according to IEC 62353

WARNING: Regardless of the accident prevention regulations or testing intervals for medical devices prescribed in different countries, with this device safety checks/repeat inspections as defined by IEC 62353 must be performed and recorded by a qualified electrician at least once a year.

Visual inspection

1. Check the device and accessories for any mechanical damage which may impair functionality.
2. Check that inscriptions relevant to safety are legible.

Electric measurements

- Inspect the device safety fuses
- Measure touch current according to IEC 62353: For limit values, please refer to the current standard.
- Measure patient leakage current according to IEC 62353: For limit values, please refer to the current standard.

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Test for proper functioning

A functional inspection must be carried out in accordance with the instruction manual.

Documentation

The safety check/repeat inspection and results must be documented.

WARNING: If any defects or shortcomings which could endanger patients, personnel or others are ascertained during these safety checks/repeat inspections, the unit must not be operated until such time as these defects or shortcomings have been eliminated by qualified technical servicing.

NOTE: For detailed information on the range and implementation of the safety check/repeat inspection, please refer to the current Service Manual.

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5.2 Servicing and repair

Defective items of equipment must be serviced and repaired exclusively by persons authorized by us; all repair work must employ original parts.

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5.3 Limitation of liability

KARL STORZ SE & Co. KG shall be liable for the failure or a deterioration in the safe operation, operational reliability, and performance of this equipment only subject to the conditions that all assembly, operations, system expansions, readjustments, modifications, or repairs to the same have been performed by persons authorized by KARL STORZ, and that the instrument has been used in accordance with its operating instructions at all times.

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5.4 Warranty

The guarantees provided can be found in the Standard Conditions of Business of KARL STORZ.

The medical device must always be sent to your local subsidiary (see 'Subsidiaries' section), even during the warranty period.

Opening the equipment or performance of any repairs or modifications to the equipment by unauthorized persons shall relieve us of any liability for its performance. Any such opening, repair, or modification performed during the warranty period shall void all warranty.

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5.5 Directive compliance

According to Medical Device Directive (MDD):

This medical device belongs to Class I
This medical device bears the CE mark in accordance with the Medical Device Directive (MDD) 93/42/EEC.

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5.6 Standard compliance

Acc. to EN 60601-1, EN 60601-2-18, UL 2601, CSA 601.1:

- Type of protection against electric shocks:
Protection Class II
- Degree of protection against electric shocks:
Applied part type BF

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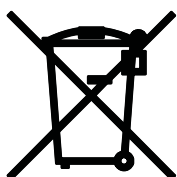
According to IEC 60601-1-2:

Please read the Electromagnetic Compatibility
Information in the appendix.

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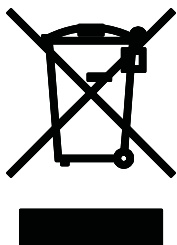
5.7 Battery

Used batteries must not be discarded as unsorted municipal waste. Consumers are obliged to return batteries to a commercial collection point or via the public waste disposal authority. You can also return the batteries to KARL STORZ. Used batteries contain harmful substances which could pose environmental or health risks.

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5.8 Disposal

This device has been marked in accordance with the European Directive on waste electrical and electronic equipment (WEEE). At the end of its useful operating life, dispose of the unit as electronic scrap. Please ask either KARL STORZ SE & Co. KG, a KARL STORZ subsidiary or your specialist dealer for information on your local collection point. Within the scope of application of this Directive, KARL STORZ SE & Co. KG is responsible for the proper disposal of this device.

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5.9 Repair program

During the repair period, you will generally receive a unit on loan, which must then be returned to KARL STORZ as soon as you receive the repaired unit.

In Germany you can refer repairs direct to:

KARL STORZ SE & Co. KG
Abt. Reparaturservice
Dr. Karl-Storz-Straße 34
78532 TUTTLINGEN, GERMANY

In all other countries please contact the respective KARL STORZ subsidiary or your specialist dealer.

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5.10 Important information

For hygiene reasons or to prevent infection, telescopes, instruments and devices must be cleaned, disinfected and, if necessary, sterilized before they are sent for repair.

We reserve the right to return contaminated instruments/devices to the sender.

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Weight 1250 g

Drip-water protection IP54

CAUTION: Drip-water protection can only be ensured if the connection sockets are sealed using the designated rubber seals.

Classification IEC Class II

Input voltage: 12 V

Current consumption: 1.5 A max.

Battery: Lithium-Ion,
3.6 V 7800 mAh

Resolution 1280 x 800

Screen width 150 mm

Screen height 94 mm

Memory card SD card

Video out signal: The connected monitor must support the 720p mode

Charging power supply unit ET27-30-0004943

Supply voltage: 100–240 VAC, 50-60 Hz

Output current: 1.5 A max.

Output voltage: 12 V

Power connection: Power adaptor for EU/UK/USA/AUS

Drip-water protection: IP40

Cable length: 2 m

Status display via LED green: Ready for operation, power supply available

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Dimensions

AC adaptor: Approx. 87 × 52 × 33 mm
(without power adaptor)

Weight Approx. 130 g

SD memory card

Data throughput: 10 MB/sec
Compatibility: Compatible with SD and SDHC cards up to 32 GB (class 4 or higher)

USB stick

Compatibility: USB 2.0-compatible, FAT-formatted, up to 32 GB

Operating/storage conditions

Operating conditions: 0 °C to +40 °C
30% to 70% rel. humidity

Charging conditions

(battery): +10 °C to +45 °C

Atmospheric pressure: 700 to 1080 hPa

Storage conditions: –10 °C to +60 °C
5% to 95% rel. humidity

Atmospheric pressure: 700 to 1080 hPa

Average lifecycle: 5 years

Open Source Software Notice

This product contains free software which falls under the terms of the General Public License (GPL) and Lesser General Public License (LGPL). If you wish to obtain the source code for this software, please contact the Technical Support via the German KARL STORZ website at www.karlstorz.com.

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6.2 Troubleshooting

WARNING: Always disconnect the device from the mains before carrying out any maintenance work!

Symptom:

Complete failure of device. Status display may be lit.

Possible causes:

- Power supply failure.
- Inadequate connection between the power cord connector and the device socket.
- System error

Remedy:

- Check that there is electricity to the wall outlet.
- Firmly push power cord connector into socket on device.
- Perform reset (see page 29).

Symptom:

No picture, TFT screen dark.

Possible causes:

- CMOS videoscope not fully inserted.
- Defective camera electronics.
- TFT screen defective.

Remedy:

- Push in the CMOS videoscope as far as it will go.
- Send the CMOS videoscope or supply unit to KARL STORZ for repair.

Symptom:

Cloudy picture, stripes, streaks, or similar.

Possible causes:

- Videoscope telescope is dirty.
- CMOS videoscope contacts are soiled.

Remedy:

- Clean with cotton swab and alcohol solution or special cleaning paste.
- Clean videoscope contacts.

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Color distortions.

Possible causes:

- White balance not carried out correctly.
- TFT screen or CMOS videoscope defective.

Remedy:

- Carry out new white balance.
- Send supply unit to KARL STORZ for repair.

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- Camera connecting cable is broken.

Remedy:

- Have a new camera connecting cable fitted.

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Symptom:

Image cannot be stored. The icon is not displayed in the menu bar.

Possible causes:

- No memory card inserted
- Memory card full
- Memory card is not recognized

Remedy:

- Insert memory card
- Change memory card
- Insert the memory card whilst the C-MAC® Monitor is switched off and then turn the monitor on again
- Perform a reset to ensure that the SD memory card is recognized.
- Format the memory card on the PC with FAT (standard).

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Abhilfe:

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Symptom:

Video stream cannot be played back on the PC monitor.

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SD memory card cannot be written on.

Possible causes:

- SD memory card is formatted incorrectly
- SD memory card is write-protected

Remedy:

- Format the SD memory card on the PC with FAT (standard).
- SD memory card may only have one partition
- Remove the write protection on the SD memory card

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USB stick cannot be written on

Possible causes:

- USB stick is formatted incorrectly.

Remedy:

- Format the USB stick on the PC with FAT (standard).
- USB stick may only have one partition.

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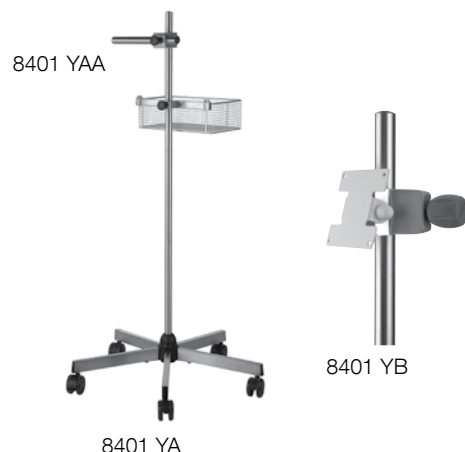
Stand, for C-MAC® Monitor, height 120 cm, star-foot roller stand with antistatic castors, crossbar 25 cm x diameter 25 mm for positioning the monitor, including storage tray for laryngoscope, measurements (w x d x h): 30 x 20 x 10 cm

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Crossbar, for stands 8401 YA, 50 cm x diameter 25 mm, for positioning the C-MAC® Monitors 8401 ZX/8402 ZX/8403 ZX

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Crossbar, for stands 8401 YA, 70 cm x diameter 25 mm, for positioning the C-MAC® Monitors 8401 ZX/8402 ZX/8403 ZX

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Clamp, VESA 75 standard, for fixation of the C-MAC® Monitor to round profile with diameter 20 – 43 mm and square profile with diameter 16 – 27 mm, for use with Monitors 8401 ZX/8402 ZX/8403 ZX

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Protective Bag, blue, for C-MAC® system, made of water-resistant and sturdy material, washable, separate compartments for the monitor and three C-MAC® video laryngoscope blades with electronic module, for use with C-MAC® Monitor 8401 ZX/8402 ZX/ 8403ZX, Electronic Module 8401 X/8402 X, Power Supply and three C-MAC® video laryngoscopes

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Airway TROLL-E trolley stand, on 4 antistatic double castors, 2 of which are lockable, with VESA 75/100 holder for mounting a monitor, integrated cable channel in the central upright, max. load capacity of monitor module 15 kg, dimensions of trolley stand: 670 x 1660 x 670 mm (W x H x D), castor diameter: 100 mm. Stand is not assembled on delivery.

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C-MAC TROLL-E, on 4 antistatic double castors, 2 of which are lockable, with stainless steel tube, dimensions of trolley stand: 670 x 1500 x 670 mm (W x H x D), castor diameter: 100 mm. Stand is not assembled on delivery.

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WARNING: Medical electrical devices are subject to special precautions regarding Electromagnetic Compatibility (EMC). Observe the EMC instructions in this Appendix during installation and operation.

Together with a CMOS videoscope, the C-MAC® Monitor 8403 ZX complies with EN/IEC 60601-1-2: [CISPR 11 Class B] and therefore meets the EMC requirements of the Medical Device Directive (MDD) 93/42/EEC.

These limits are designed to provide reasonable protection against the typical electromagnetic interference to be expected in a medical environment. The above-mentioned devices/combinations are Group 1 devices (acc. to CISPR 11).

Group 1 contains all the 'equipment and systems which use or generate RF energy exclusively for internal functions'.

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NOTE: The tables and guidelines that are included in this Appendix provide information to the customer or user that is essential in determining the suitability of the equipment or system for the electromagnetic environment of use, and in managing the electromagnetic environment of use to permit the equipment or system to perform its intended use without disturbing other equipment and systems or non-medical electrical equipment. If this device does cause electromagnetic interferences with other devices, the user is encouraged to try to correct the interferences by one or more of the following measures:

- reorient or relocate the receiving device;
- increase the separation between the individual devices;
- connect the devices to different electrical circuits.

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WARNING: The use of portable and mobile RF equipment may have an impact on this and other pieces of medical equipment.

WARNING: The use of accessories other than those specified in the KARL STORZ instruction manual may result in increased emissions or decreased interference immunity of the C-MAC® Monitor 8403 ZX in combination with a CMOS videoscope. The accessories listed below have been shown to comply with the requirements of EN/IEC 60601-1-2. When using accessories other than those specified here, it is the responsibility of the operator to ensure that they comply with EN/IEC 60601-1-2.

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Accessories and cables which have been shown to comply with EN/IEC 60601-1-2: 2001:

Type	Shield	Length [m]	Ferrite	Used for
Charger ET27-30-0004943	No	2.0	No	Charging the internal battery of the C-MAC® Monitor 8403 ZX

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Table 1 Guidelines and Manufacturer's Declaration – Electromagnetic Emissions		
The Monitor 8403 ZX in combination with a CMOS videoscope is designed for operation in an environment as defined below. The user of the device should make sure that the system is operated in such an environment.		
Interference emission measurements	Conformity level	Electromagnetic environment – guidelines
RF emissions acc. to CISPR 11	Complies with Class B	The above-mentioned devices or combinations of devices use RF energy exclusively for internal functions. Therefore, their RF emissions are very low and are not likely to cause interference in nearby electronic equipment.
RF emissions acc. to CISPR 11	Complies with Class B	The above-mentioned devices or combinations of devices are suitable for use in residential environments.
Emission of harmonic oscillations acc. to IEC 61000-3-2	Class A. Not applicable	
Voltage fluctuations/flicker emissions acc. to IEC 61000-3-3	Compliant	

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Table 2
Guidelines and Manufacturer's Declaration – Electromagnetic Immunity

The Monitor 8403 ZX in combination with a CMOS videoscope is designed for operation in an environment as defined below. The user of the device should make sure that the system is operated in such an environment.

Immunity tests	EN/IEC 60601 test level	Compliance level	Electromagnetic environment - guidelines
Electrostatic discharge (ESD) acc. to IEC 61000-4-2	± 6 kV contact discharge ± 8 kV air discharge	Complies ± 6 kV contact discharge ± 8 kV air discharge	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity must be at least 30%.
Rapid transient electrical interferences/bursts acc. to IEC 61000-4-4	± 2 kV for power lines ± 1 kV for input and output lines	Complies ± 2 kV for power lines ± 1 kV for input and output lines	Mains power quality should be that of a typical commercial or hospital environment. If image distortion occurs it may be necessary to further filter the power quality by using an AC line filter.
Surges acc. to IEC 61000-4-5	± 1 kV differential mode ± 2 kV common mode	Complies ± 1 kV differential mode ± 2 kV common mode	Mains power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and fluctuations of the supply voltage acc. to IEC 61000-4-11	<5 % U_T * (>95 % dip in U_T) for ½ cycle 40 % U_T (60 % dip in U_T) for 5 cycles 70 % U_T (30 % dip in U_T) for 25 cycles <5 % U_T (>95 % dip in U_T) for 5 seconds	Complies <5 % U_T * (>95 % dip in U_T) for ½ cycle Complies 40 % U_T (60 % dip in U_T) for 5 cycles Complies 70 % U_T (30 % dip in U_T) for 25 cycles Complies <5 % U_T (>95 % dip in U_T) for 5 seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the equipment requires continued operation even during interruptions to the power supply, it is recommended that the equipment or system be powered from an uninterruptible power supply.
Magnetic field at the supply frequency (50/60 Hz) acc. to IEC 61000-4-8	3 A/m	Complies 3 A/m	The magnetic fields at the mains frequency should be at the values typical of commercial or hospital environments.

* Note: U_T is the mains alternating voltage before applying the test levels.

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Table 4
Guidelines and Manufacturer's Declaration – Electromagnetic Immunity –
for medical electrical devices that are not life-supporting

The Monitor 8403 ZX in combination with a CMOS videoscope is designed for operation in an environment as defined below.
 The user of the device should make sure that the system is operated in such an environment.

Immunity tests	EN/IEC 60601 test level	Compliance level	Electromagnetic environment – guidelines
			Portable and mobile RF communications equipment should be used no closer to any part of the above-mentioned devices or device combinations, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distances:
Conducted RF disturbances acc. to IEC 61000-4-6	3 V _{eff} 150 kHz to 80 MHz	3 V _{eff}	$d = [3.5/3]\sqrt{P}$ 150 kHz to <80 MHz
Radiated RF disturbances acc. to IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	3 V/m	$d = [3.5/3]\sqrt{P}$ 80 MHz to 800 MHz $d = [7/3]\sqrt{P}$ 800 MHz to 2.5 GHz
			Where P is the nominal power of the transmitter in watts [W] according to the information provided by the manufacturer of the transmitter and d is the recommended separation distance in meters [m]. The field strength of stationary transmitters at all frequencies on site ^{a)} should be, according to a study, less than the conformity level ^{b)} . Interference may occur in the vicinity of equipment marked with the following symbol: 

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Note 2: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of buildings, objects, and persons.

^{a)} Field strengths from fixed transmitters, such as base stations for radio [cellular/cordless] telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the unit is used exceeds the applicable RF compliance level above, the unit should be observed to verify normal operation. If unusual performance characteristics are observed, additional measures may be necessary, such as a modified alignment or a different location for the device.

^{b)} Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 10 V/m.

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Table 6
Recommended separation distances between portable and mobile RF communications equipment and the KARL STORZ CMOS videoscope

The Monitor 8403 ZX in combination with a CMOS videoscope is designed for operation in an electromagnetic environment in which the RF disturbances are controlled. The user of the device can help to avoid electromagnetic interferences by observing the minimum distance between portable and mobile RF telecommunication devices (transmitters) and the device – as a function of the output power of the communication device, as shown below.

Rated maximum output power of transmitter [W]	Separation distance d [m] as a function of the transmitter frequency		
	150 kHz to 80 MHz $d = [3.5/3]\sqrt{P}$	80 MHz to 800 MHz $d = [3.5/3]\sqrt{P}$	800 MHz to 2.5 GHz $d = [7/3]\sqrt{P}$
0.01	0.1	0.1	0.2
0.1	0.4	0.4	0.7
1	1.2	1.2	2.3
10	3.7	3.7	7.4

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation belonging to the relevant column, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.

Note 2: These guidelines may not apply in all situations. The propagation of electromagnetic quantities is affected by absorptions and reflections of structures, objects and people.

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KARL STORZ Endoscopy Canada Ltd.
7171 Millcreek Drive, Mississauga
Ontario, L5N 3R3, Canada
Phone: +1 905 816-4500, Fax: +1 905 858-4599
Toll free phone: 1-800-268-4880 (Canada only)
Toll free fax: 1-800-482-4198 (Canada only)
E-Mail: info.canada@karlstorz.com

KARL STORZ Endoscopy-America, Inc.
2151 East Grand Avenue
El Segundo, CA 90245-5017, USA
Phone: +1 424 218-8100, Fax: +1 424 218-8526
Toll free phone: 800 421-0837 (USA only)
Toll free fax: 800 321-1304 (USA only)
E-Mail: info@kseas.com

KARL STORZ Veterinary Endoscopy-America, Inc.
1 South Los Carneros Road
Goleta, CA 93117, USA
Phone: +1 805 968-7776, Fax: +1 805 685-2588
E-Mail: info@karlstorvet.com

KARL STORZ Endoscopia Latino-America, Inc.
815 N. W. 57th Avenue, Suite 480
Miami, FL 33126-2042, USA
Phone: +1 305 262-8980, Fax: +1 305 262-8986
E-Mail: info@kseas.com

KARL STORZ Endoscopia México S.A. de C.V.
Edificio Atlantic, Oficina 3G
Calle D e/ 1ra y 3ra
10400 Vedado, Havana, Cuba
Phone: +537 836 95 06, Fax: +537 836 97 76
E-Mail: kstorzucuba@gmail.com

KARL STORZ Endoscopia México S.A. de C.V.
Lago Constanza No 326,
Col. Granada Del. Miguel Hidalgo
C.P. 11520 México D.F.
Phone: +52 (55) 1101 1520
E-Mail: mx-info@karlstorz.com

KARL STORZ Marketing América Do Sul Ltda.
Rua Joaquim Floriano, nº. 413, 20º andar – Itaim Bibi,
CEP-04534-011 São Paulo, Brasil
Phone: +55 11 3526-4600, Fax: +55 11 3526-4680
E-Mail: info@karlstorz.com.br

KARL STORZ Endoscopia Argentina S.A.
Zufriategui 627 6° Piso, B1638 CAA - Vicente Lopez
Provincia de Buenos Aires, Argentina
Phone: +54 11 4718 0919, Fax: +54 11 4718 2773
E-Mail: info@karlstorz.com.ar

KARL STORZ Endoskopi Norge AS
Østsjøveien 15B
0661 Oslo, Norway
Phone: +47 6380 5600, Fax: +47 6380 5601
post@karlstorz.no

KARL STORZ Endoskop Sverige AB
Storsåtragränd 14
12739 Skärholmen, Sweden
14108 Kungens Kurva, Sweden
Phone: +46 8 505 648 00
E-Mail: kundservice@karlstorz.se
KARL STORZ Endoscopy Suomi OY
Taivaltie 5
01610 Vantaa, Finland
Phone: +358 (0)96824774, Fax: +358 (0)968247755
E-Mail: asiakaspalvelu@karlstorz.fi

KARL STORZ SE & Co. KG
Representation Office
Kestucio g. 59
81224 Vilnius, Lithuania
Phone: +370 68567000
E-Mail: LT-LV-info@karlstorz.com

KARL STORZ Endoskopi Danmark A/S
Skovlytoften 33
2840 Holte, Denmark
Phone: +45 45162600, Fax: +45 45162609
E-Mail: marketing@karlstorz.dk

KARL STORZ Endoscopy (UK) Ltd.
415 Perth Avenue, Slough
Berkshire, SL1 4TQ, United Kingdom
Phone: +44 1753 503500, Fax: +44 1753 578124
E-Mail: info-uk@karlstorz.com

KARL STORZ Endoscopie Nederland B. V.
Displayweg 2
3821 BT Amersfoort, Netherlands
Phone: +31 (0)33 4545890
E-Mail: info-nl@karlstorz.com

KARL STORZ Endoscopy Belgium N. V.
Phone: +31 (0)33 4545890
E-Mail: info-be@karlstorz.com

KARL STORZ Endoscopie France S. A. S.
12, rue Georges Gynemer, Quartier de l'Europe
78280 Guyancourt, France
Phone: +33 1 30484200, Fax: +33 1 30484201
E-Mail: marketing@karlstorz.fr

KARL STORZ Endoskop Austria GmbH
Landstraßer Hauptstr. 148/1/G1
1030 Wien, Austria
Phone: +43 1 71 56 0470, Fax: +43 1 71 56 0479
E-Mail: storz-austria@karlstorz.at

KARL STORZ Endoscopia Ibrica S. A.
Parque Empresarial San Fernando
Edificio Munich – Planta Baja
28830 Madrid, Spain
Phone: +34 91 6771051, Fax: +34 91 6772981
E-Mail: info-es@karlstorz.com

KARL STORZ Endoscopia Italia S. r. l.
Via dell'Artigianato, 3
37135 Verona, Italy
Phone: +39 045 8222000, Fax: +39 045 8222001
E-Mail: info-ita@karlstorz.com

KARL STORZ Adria Eos d.o.o.
Capraška 6
10000 Zagreb, Croatia
Phone: +385 1 6406 070, Fax: +385 1 6406 077
E-Mail: info@karlstorz.hr

KARL STORZ Endoskopija d.o.o.
Verovškova c. 60A
1000 Ljubljana, Slovenia
Phone: +386 1 620 5880, Fax: +386 1 620 5882
E-Mail: pisarna@karlstorz.si

KARL STORZ Endoskop Polska Marketing Sp. z o.o.
ul. Bojkowska 47
44-100 Gliwice, Poland
Phone: +48 32 706 13 00, Fax: +48 32 706 13 07
E-Mail: info-pl@karlstorz.com

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KARL STORZ ENDOSKOPE
East Mediterranean and Gulf S.A.L.
Block M, 3rd Floor
Beirut Souks, Weygand Street
2012 3301 Beirut, Lebanon
Phone: +961 1 999390, Fax: +961 1 999391
E-Mail: info@karlstorz-emg.com

KARL STORZ Endoscopy (South Africa) (Pty) Ltd.
P.O. 6061, Roggebaai 8012
Cape Town, South Africa
Phone: +27 21 417 2600, Fax: +27 21 421 5103
E-Mail: info@karlstorz.co.za

TOO KARL STORZ Endoscopy Kasachstan
6, Saryarka str. BC „Arman“, office 314
010000 Astana, Republic of Kazakhstan
Phone: +7 7172 552-549, 552-788, Fax: -444
E-Mail: info@karlstorz.kz

KARL STORZ ENDOSKOPE
East Mediterranean & Gulf (branch)
Building West Side 7A – Unit 7WA – 3008
Dubai Airport Free Zone, P.O. Box 54983
Dubai – United Arab Emirates
Phone: +971 (0)4 2958887, Fax: +971 (0)4 3205282
Service Hotline: +971 (0)4 3415882
E-Mail: info-gne@karlstorz-emg.com

KARL STORZ Endoscopy India Private Limited
11th Floor, Dr. Gopal Das Bhawan
28, Barakhamba Road
New Delhi 110001, India
Phone: +91 11 4374 3000, Fax: +91 11 4374 3010
E-Mail: corporate@karlstorz.in

KARL STORZ SE & Co. KG
Resident Representative Office
14th Floor, M Plaza Saigon
39 Le Duan, District 1
Ho Chi Minh City, Vietnam
Phone: +84 8 3823 8000, Fax: +84 8 3823 8039
E-Mail: infovietnam@karlstorz.com

KARL STORZ Endoscopy Australia Pty. Ltd.
15 Orion Road Lane Cove NSW 2066
P O Box 50 Lane Cove NSW 1595, Australia
Phone: +61 (0)2 9490 6700
Toll free: 1800 996 562 (Australia only)
Fax: +61 (0)2 9420 0695
E-Mail: info@karlstorz.au

KARL STORZ Endoscopy China Ltd., Hong Kong
Room 2503-05, 25F AXA Tower, Landmark East,
No. 100 Hing Ming Street, Kwun Tong, Kowloon,
Hong Kong, People's Republic of China
Phone: +852 28 65 2411, Fax: +852 28 65 4114
E-Mail: inquiry@karlstorz.com.hk

KARL STORZ Endoscopy (Shanghai) Ltd.
Beijing Branch,
Room 1805-1807, Building B, 18F Beijing IFC
No. 8, Jianguomenwai Street, Chaoyang District,
Beijing, 100022, People's Republic of China
Phone: +86 10 5638188, Fax: +86 10 5638199
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Shanghai Branch
36F, Tower 1 Grand Gateway
No. 1 Hongqiao Road, Xuhui District,
Shanghai, 200030, People's Republic of China
Phone: +86 21 6113-1188, Fax: +86 21 6113-1199
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Chengdu Branch
Room 805, 8F Jin Jiang International Building
No. 1 West Linjiang Road, Wuhou District,
Chengdu, 610044, People's Republic of China
Phone: +86 28 86587977, Fax: +86 28 86587975
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Shenyang Branch,
Room 2001-2005, 20F N-MEDIA International Center,
No. 167 Youth Avenue, Shenghe District,
Shenyang, 110014, People's Republic of China
Phone: +86 24 23181118, Fax: +86 24 23181119
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy (Shanghai) Ltd.
Guangzhou Branch
Room 02B, 35F Teem Tower,
No. 208 Tianhe Road, Tianhe District,
Guangzhou, 510620, People's Republic of China
Phone: +86 20 8732-1281, Fax: +86 20 8732-1286
E-Mail: info@karlstorz.com.cn

KARL STORZ Endoscopy Asia Marketing Pte Ltd.
No. 8 Commonwealth Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com.sg

KARL STORZ Endoscopy Singapore Sales Pte Ltd
No. 8 Commonwealth Lane #03-02
Singapore 149555, Singapore
Phone: +65 69229150, Fax: +65 69229155
E-Mail: infoasia@karlstorz.com.sg

KARL STORZ Endoscopy Korea Co. Ltd.
I-Park Bundang1, 102dong, 8F
239 Jeongjail-Ro, Bundang-Gu
Seongnam, 13556, Korea
Phone: +82-70-4350-7474, Fax: +82-70-8277-3299
E-Mail: infokorea@karlstorz.com

KARL STORZ Endoscopy Taiwan Ltd.
12F, No. 192, Sec. 2, Chung Hsin Rd.,
Sindian District, New Taipei City, Taiwan
Phone: +886 933 014 160, Fax: +886 2 8672 6399
E-Mail: mingwang@karlstorz.tw

KARL STORZ SE & Co. KG
Representative Office Philippines
1901 Picadilly Star Bldg., 4th Avenue, BGC
Taguig City 1636, Philippines
Phone: , Fax: +63 2 31745-00
E-Mail: philippines@karlstorz.com

KARL STORZ Endoscopy Japan K. K.
Stage Bldg. 8F, 2-7-2 Fujimi
Chiyoda-ku, Tokyo 102-0071, Japan
Phone: +81 3 6380-8622, Fax: +81 3 6380-8633
E-Mail: info@karlstorz.co.jp

www.karlstorz.com

KARL STORZ SE & Co. KG

Dr.-Karl-Storz-Straße 34
78532 Tuttlingen

Postfach 230
78503 Tuttlingen
Germany

Telefon: +49 (0)7461 708-0
Telefax: +49 (0)7461 708-105
E-Mail: info@karlstorz.com
Web: www.karlstorz.com