

New Protection Cap for the C-MAC® system interface



The C-MAC® system interface (connector 1) has been validated without protection cap and can be manually or machine cleaned and disinfected without using a protection cap. Particularly in manual reprocessing, subsequent thorough rinsing with distilled and microbiologically pure/sterile water must be ensured. This also applies to the plug (connector 2) of the C-MAC® connecting cable or the C-MAC® PM, and the LEMO plug (connector 3) of the FIVE or the C-MAC® connecting cable. As the last step, the connections should always be dried with medical compressed air to ensure that the C-MAC® system interface and the C-MAC® connecting cable or the C-MAC® PM are not connected in wet condition. The new KARL STORZ Protection Cap 8403 YZ is available if this step cannot be ensured at all times or if hospital procedures require a protection cap.

The reusable protection cap must be dry when placed on the respective interface and the plugs.

Special Features:

- Suitable for all endoscopes with C-MAC® system interface or endoscopes/cables with LEMO plug
- Three connections (connector 1, 2, and 3) for universal use
- Reusable



PRODUCT INFO

**ANESTHESIOLOGY
AND EMERGENCY
MEDICINE**

*Confidential –
for internal
use only!*

Connection options:

1. Connection to C-MAC® video laryngoscopes, C-MAC® S IMAGER, or C-MAC® VS:

Connector 1 of the protection cap is connected to the C-MAC® system interface of the C-MAC® Video Laryngoscopes (8403 xxx), C-MAC® S IMAGER (8403 XSI/ XSP), or C-MAC® VS (10331 BX).



2. Connection to the C-MAC® PM or C-MAC® connecting cable:

Connector 2 of the protection cap is connected to the C-MAC® PM (8403 XD) or the C-MAC® Connecting Cable (8403 X).



3. Connection to the LEMO plug of the C-MAC® connecting cable or the Flexible Intubation Video Endoscopes (FIVE):

Connector 3 of the protection cap is connected to the LEMO plug of the C-MAC® Connecting Cable (8403 X) or the FIVEs (11301 ABX, 11302 BDX, 11301 BNX).



PRODUCT INFO

**ANESTHESIOLOGY
AND EMERGENCY
MEDICINE**

*Confidential –
for internal
use only!*

The new protection cap features two markings:

A dot at connector 2 and an arrow at connector 3 for the LEMO plug. They help to optimally position the protection cap and to prevent incorrect application.

Protection cap for C-MAC® system interface



8403 YZ

Protection Cap, for the C-MAC® system interface on C-MAC® Video Laryngoscopes 8403 xxx and C-MAC® PM 8403 XD as well as C-MAC® Connecting Cable 8403 X, to protect plug contact during reprocessing, cap is reusable, for use with C-MAC® Connecting Cable 8403 X, C-MAC® POCKET MONITOR 8403 XD, C-MAC® Video Laryngoscopes 8403 xxx, Electronic Modules 8401 X / 8402 X, C-MAC® S IMAGER 8403 XSI, C-MAC® S Pediatric IMAGER 8403 XSP and Flexible Intubation Video Endoscopes (FIVE)

Availability: As of October 2017



PRODUCT INFO

**ANESTHESIOLOGY
AND EMERGENCY
MEDICINE**

*Confidential -
for internal
use only!*